



4-PLY ASTM LEVEL 3 SURGICAL DISPOSABLE FACE MASK

COMFORTABLE LIGHTWEIGHT MATERIAL
MADE IN NEW YORK



INDICATIONS FOR USE

This 4-Ply Surgical Mask is intended to be worn to protect both the patient and healthcare professional from transfer of microorganisms, body fluids, and particulate material. This 4-Ply Surgical Mask is intended for use in infection control practices to reduce the potential exposure to blood and body fluids. This is a single use, non-sterile, disposable device.

NOTE

Surgical masks may be effective in blocking splashes and large-particle droplets; however, because of the loose fit between the surface of the surgical mask and the user's face, leakage can occur around the edge of the mask when the user inhales. Therefore, a surgical mask may not provide the user with a reliable level of protection from inhaling smaller airborne particles and is not considered respiratory protection. For this reason, surgical masks are not recommended for use in aerosol generating procedures and any clinical conditions where there is significant risk of infection through inhalation exposure. In such clinical conditions, a filtering facepiece respirator (such as an N95 respirator) with a tight fit is recommended to provide a more reliable level of respiratory protection against pathogenic biologic airborne particulates.

INSTRUCTIONS FOR USE

1. Remove mask from box.
2. Inspect Mask. Ensure no tears or holes in mask.
3. Blue color faces outward and nose clip on top.
4. Place ear loops over ears.
5. Pinch down on nose clip with clean hands.
6. Adjust mask to fit under chin.
7. Ensure mask fits firmly to your face.
8. To remove the mask, carefully unhook from the ears and pull away from face without touching the front.

CONTRADICTIONS, WARNINGS AND PRECAUTIONS

- This 4-Ply Surgical Mask is a single use, disposable device, and is provided non-sterile.
- DO NOT attempt to clean, sterilize, or disinfect the mask after use.
- This 4-Ply Disposable Surgical Mask is not intended to provide protection against pathogenic biological airborne particulates and are not recommended for use in aerosol generating procedures and any clinical conditions where there is significant risk of infection through inhalation exposure.
- This 4-Ply Surgical Masks does not contain any drugs, biologics, nanoparticles or antimicrobial/antiviral agents.
- The materials that are body contacting in this 4-Ply Disposable Surgical Mask are as follows:
 - Outer Mask Layer – Spunbond nonwoven polypropylene.
 - Inner Mask Layer – Spunbond nonwoven polypropylene.
 - Middle Mask Layer – Melt blown nonwoven polypropylene
 - Ear Loops – Nylon / Spandex
 - Nose Wire – Virgin polyethylene plastic, 24 gauge soft annealed carbon steel, and kraft paper (nominal basis weight of 25 lbs. /ream)



ALTOR SAFETY

845.422.8320 · altorsafety.com · 711 Executive Boulevard, Suite C, Valley Cottage NY 10989

